

◀ TEAR HERE

TEAR HERE ▶

**For Temporary External Use  
To Control Traumatic Bleeding**  
ONE (1) 3 IN X 4 FT (7.5 CM X 1.2 M) HEMOSTATIC DRESSING



Made in the USA

REORDER 475

◀ TEAR HERE

TEAR HERE ▶

## DIRECTIONS FOR USE



**1.** Open package and remove Rolled Gauze. Keep the empty package.



**2.** Pack Rolled Gauze into wound and use it to apply pressure directly over bleeding source. More than one Rolled Gauze may be required.



**3.** Continue to apply pressure for 3 minutes or until bleeding stops.



**4.** Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

**PRODUCT REMOVAL:** 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

**Manufactured by:**

Z-Medica®, LLC  
4 Fairfield Boulevard  
Wallingford, CT 06492 USA  
Tel: 1.877.750.0504, +1.203.294.0000  
Fax: 1.800.343.8656, +1.203.303.7216  
QuikClot.com

Patents: Z-Medica.com/patents.aspx

© 2020 Z-Medica, LLC

All rights reserved.

Contains kaolin.

101311-01-WEB Rev I 10/9/2020



2797

EC REP

Emergo Europe  
Prinsessegracht 20  
2514 AP  
The Hague  
The Netherlands

## FOR USE BY TRAINED EMERGENCY RESPONDERS



Do Not Reuse



**WARNINGS:** For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes. Reuse will cause risk of infection and loss of efficacy.



Sterile Barrier System

STERILE R

MD Medical Device