

QuikClot®

COMBAT



GAUZE[®] LE

Vacuum Packed and Z-Folded

For Temporary External Use
To Control Traumatic Bleeding

ONE (1) 3 IN X 4 YDS (7.5 CM X 3.7 M) HEMOSTATIC DRESSING



DIRECTIONS FOR USE



1. Open package and remove **Combat Gauze LE**. Keep the empty package.



2. Pack **Combat Gauze LE** into wound and use it to apply pressure directly over bleeding source. More than one **Combat Gauze LE** may be required.



3. Continue to apply pressure for 3 minutes or until bleeding stops.



4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

CE
2797

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP
The Hague
The Netherlands

REF 350

PRODUCT REMOVAL: 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

Manufactured by:

Z-Medica®, LLC

4 Fairfield Boulevard

Wallingford, CT 06492 USA

Tel: 1.877.750.0504, +1.203.294.0000

Fax: 1.800.343.8656, +1.203.303.7216

QuikClot.com

Patents: Z-Medica.com/patents.aspx

© 2020 Z-Medica, LLC

All rights reserved.

Contains kaolin.

100885-02-WEB Rev D 10/8/2020



Made in the USA

FOR USE BY TRAINED EMERGENCY RESPONDERS



WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes. Reuse will cause risk of infection and loss of efficacy.